Manufacturer Výrobce Product Gyártó Výrobca Proizvajalec Proizvodac	EC REP EU-representative Przedstawiciel na terenie Unii Europejskiej Importer in EU Dovozce do EU Importer na terenie UE EU importer Dovozca pre EU Uvoznik v EU Uvoznik u EU	Production facility Výrobní závod Zaklad produkcyjny Gyártás helyszíne Výrobný podnik Proizvodni pogon MANUFACTURING VIETNAM CO., LTD. No 28 VSIP II, Street 2, Vietnam-Singapore Industrial Park II, Binh Duong Industry-Service-Urban Complex, Hoa Phu Ward, Thu Dau Mot City, Binh Duong Province, Vietnam OMRON HEALTHCARE OMRON HEALTHCARE UK LTD. Opal Drive, Fox Milne, Milton Keynes, MK15 0DG, UK www.omron-healthcare.com/distributors	Subsidiaries Przedsiębiorstwa zależne Decifine spoločnosti Leányvállalatok Pobocky Hétfiráske družbe Podružnice	Importer in the United Kingdom and UK responsible person www.omron-healthcare.com/distributors OMRON MEDIZINTECHNIK HANDELSGESELLSCHAFT mbH www.omron-healthcare.com/distributors OMRON SANTÉ FRANCE SAS www.omron-healthcare.com/distributors	Distributeur a servis pro ČR CELLIMED® CELLIMED s.r.o. Sociální pčče 3487/5a, 400 11 Ústí nad Labem podružní servis - tel.: 475 208 180, e-mail: info@cellimed.cz, www.cellimed.cz

M2+ Connect (HEM-7188T1-LE)

Automatic Upper Arm Blood Pressure Monitor

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Instruction Manual 1: Safety and Other Information

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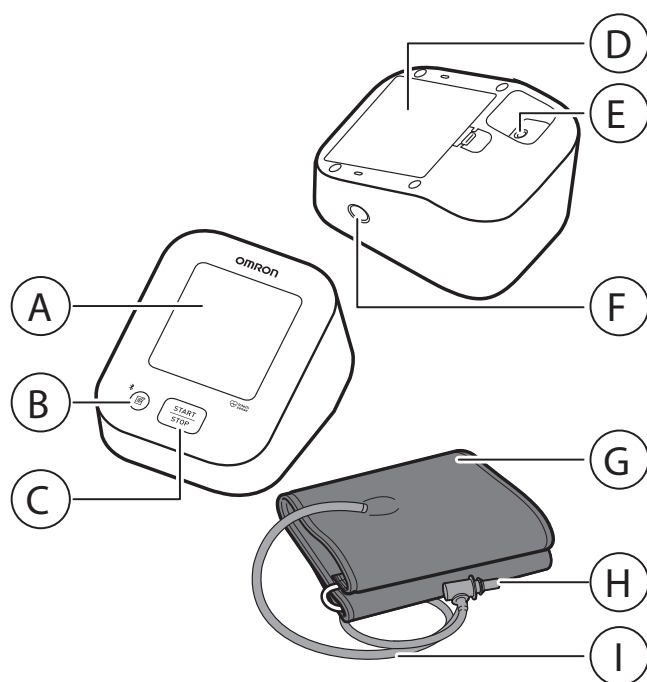
<https://www.omron-healthcare.com/>

Introduction

Thank you for purchasing the OMRON Automatic Upper Arm Blood Pressure Monitor. This blood pressure monitor uses the oscillometric method of blood pressure measurement. This means this monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

Safety Instructions

This instruction manual provides you with important information about the OMRON Automatic Upper Arm Blood Pressure Monitor. To ensure the safe and proper use of this monitor, READ and UNDERSTAND all of these instructions. **If you do not understand these instructions or have any questions, contact your OMRON retail outlet or distributor before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.**

Intended Use

Intended Purpose

This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patient population. The device detects the appearance of irregular heartbeats during measurement and indicates this via a symbol with the measurement result.

Intended Patients

Adult patient population

Intended Users

Adult population (may include patients themselves) who can understand this instruction manual.

Clinical Benefit

Patient's blood pressure can be measured non-invasively and simply in the home environment.

Type of Use

This monitor is intended to be multiple patient multiple use.

Limitation

Patient's arm circumference must be 17–42 cm.

Indication

This device is used by healthy individuals, patients with hypertension, patients with health-conscious individuals, in a general household situation for measuring blood pressure and pulse rate.

Receiving and Inspection

Remove this monitor and other components from the packaging and inspect for damage. If this monitor or any other components is damaged, DO NOT USE and consult with your OMRON retail outlet or distributor.

Important Safety Information

Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your safety.

Keep for future reference. **For specific information**

about your own blood pressure, consult with your physician.

Contraindications

- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion.
- DO NOT use this monitor on infants, toddlers, children or persons who cannot express themselves.

Side Effects

- Taking measurements more often than necessary may cause bruising due to blood flow interference.
- Inflating to a higher pressure than necessary may result in bruising of the arm where the cuff is applied. NOTE: refer to "If your systolic pressure is more than 210 mm Hg" in the end of instruction manual 2 for additional information.
- Stop using this monitor and consult with your physician if you experience skin irritation or discomfort.

Warning

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT adjust medication (including altering your use of any drug or treatment) based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified

to diagnose and treat high blood pressure and heart related conditions.

- NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.
- If you are experiencing any symptoms or concerns, consult with your physician.
- DO NOT postpone/stop regular checkups or physician visits based on the results that you are getting from this monitor.
- DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT use this monitor in oxygen rich environments or near flammable gas.
- Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions in addition to patient motion, trembling, or shivering may affect the measurement reading.
- To help avoid strangulation, keep the air tube and AC adapter cable away from infants, toddlers and children.
- This product contains small parts that may cause a choking hazard if swallowed by infants, toddlers and children.

Poznámky

Do not throw the air plug away. The air plug can be applicable to the optional cuff.

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Battery Handling and Usage

- Keep the batteries out of the reach of infants, toddlers and children.

Data Transmission

- This product always emits radio frequencies (RF) in the 2.4 GHz band. DO NOT use this product in locations where RF is restricted, such as on an aircraft or in hospitals. Disable the **Bluetooth®** feature in this monitor, or remove the batteries and unplug the AC adapter when in RF restricted areas.

AC Adapter (optional accessory) Handling and Usage

- DO NOT use the AC adapter if this monitor or the AC adapter cable is damaged. If this monitor or the cable is damaged, turn off the power and unplug the AC adapter immediately.
- Plug the AC adapter into the appropriate voltage outlet. DO NOT use in a multi-outlet plug.
- NEVER plug in or unplug the AC adapter from the electric outlet with wet hands.
- DO NOT disassemble or attempt to repair the AC adapter.

Battery Handling and Usage

- Keep the batteries out of the reach of infants, toddlers and children.

Precaution

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment or other property

- Consult with your physician before using this monitor on an arm where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present because of temporary interference to blood flow and could result in injury.
- Consult with your physician before using this monitor if you have had a mastectomy or lymph node clearance.
- Consult with your physician before using this monitor if you have severe blood flow problems or blood disorders as cuff inflation can cause bruising.
- ONLY inflate the arm cuff when it is applied on your upper arm.
- Remove the arm cuff if it does not start deflating during a measurement.
- DO NOT use this monitor for any purpose other than measuring blood pressure.
- During measurement, make sure that no mobile device or any other electrical device that emits electromagnetic fields is within 30 cm of this monitor. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT disassemble or attempt to repair this monitor.

tor or other components. This may cause an inaccurate reading.

- DO NOT use in a location where there is moisture or a risk of water splashing this monitor. This may damage this monitor.
- DO NOT use this monitor in a moving vehicle such as in a car or on an aircraft.
- DO NOT drop or subject this monitor to strong shocks or vibrations.
- DO NOT use this monitor in places with high or low humidity or high or low temperatures. Refer to section 4.
- During measurement, observe the arm to ensure that the monitor is not causing prolonged impairment to blood circulation.
- DO NOT use this monitor with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation of the devices and/or cause an inaccurate reading.
- Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for at least 30 minutes before taking a measurement.
- Rest for at least 5 minutes before taking a measurement.
- Remove tight-fitting or thick clothing from your arm while taking a measurement.
- Remain still and DO NOT talk while taking a measurement.
- ONLY use the arm cuff on persons whose arm circumference is within the specified range of the cuff.

- Ensure that this monitor has acclimated to room temperature before taking a measurement. Taking a measurement after an extreme temperature change could lead to an inaccurate reading. It is recommended that you wait for approximately 2 hours for the monitor to warm up or cool down when the monitor is used in an environment within the temperature specified as operating conditions after it is stored either at the maximum or at the minimum storage temperature. For additional information on operating and storage / transport temperature, refer to section 4.
- DO NOT use this monitor after the durable period has ended. Refer to section 4.
- DO NOT crease the arm cuff or the air tube excessively.
- DO NOT fold or kink the air tube while taking a measurement. This may cause an injury by interrupting blood flow.
- To unplug the air plug, pull on the plastic air plug at the base of the tube, not the tube itself.
- ONLY use the AC adapter, arm cuff, batteries and accessories specified for this monitor. Use of unsupported AC adapters, arm cuffs and batteries may damage and/or may be hazardous to this monitor.
- ONLY use the approved arm cuff for this monitor. Use of other arm cuffs may result in incorrect readings.
- Read and follow the "Correct Disposal of This Product" in section 5 when disposing of the device and any used accessories or optional parts.

If your systolic pressure is more than 210 mmHg: After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure. Do not inflate above 299 mmHg.

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
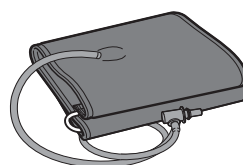
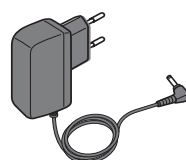
**Sítový adaptér
Model: HHP-CM01**

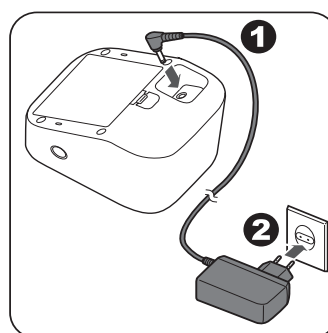Data Transmission

- DO NOT replace the batteries or unplug the AC adapter while your readings are being transferred to your smart device. This may result in incorrect operation of this monitor and failure to transfer your blood pressure data.

AC Adapter (optional accessory) Handling and Usage

- Fully insert the AC adapter into the outlet.
- When unplugging the AC adapter from the outlet, be sure to safely pull from the AC adapter. DO NOT pull from the AC adapter cable.
- When handling the AC adapter cable: DO NOT damage it. / DO NOT break it. / DO NOT tamper with it. / DO NOT pinch it. / DO NOT forcibly bend or pull it. / DO NOT twist it. / DO NOT use it if it is gathered in a bundle. / DO NOT place it under heavy objects.
- Wipe any dust off of the AC adapter.
- Unplug the AC adapter when not in use.
- Unplug the AC adapter before wiping this monitor.

Battery Handling and Usage

- DO NOT insert batteries with their polarities incorrectly aligned.
- ONLY use 4 "AA" alkaline or manganese batteries with this monitor. DO NOT use other types of batteries. DO NOT use new and used batteries together. DO NOT use different brands of batteries together.

- Remove the batteries if this monitor will not be used for a long period of time.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult with your physician immediately.
- If battery fluid should get on your skin, wash your skin immediately with plenty of clean, lukewarm water. If irritation, injury or pain persists, consult with your physician.
- DO NOT use batteries after their expiration date.
- Periodically check the batteries to ensure they are in good working condition.

General Notices

- To stop a measurement, press the [START/STOP] button while taking a measurement.
- When you take a measurement on the right arm, the air tube should be at the side of your elbow. Be careful not to rest your arm on the air tube.

1 Error Messages and Troubleshooting

If any of the below problems occur during measurement, check to make sure that no other electrical device is within 30 cm of the monitor. If the problem persists, refer to the table below.

EN

Display/Problem	Possible Cause	Solution
E1 appears or the arm cuff does not inflate.	The [START/STOP] button was pressed while the arm cuff is not applied.	Press the [START/STOP] button again to turn the monitor off.
	Air plug is not completely plugged into the monitor.	Insert the air plug securely.
	The arm cuff is not applied correctly.	Apply the arm cuff correctly, then take another measurement. Refer to section 6 of instruction manual 2.
	Air is leaking from the arm cuff.	Replace the arm cuff with a new one. Refer to section 13 of instruction manual 2.
E2 appears or a measurement cannot be completed after the arm cuff inflates.	Moving or talking during a measurement causes the arm cuff to not inflate sufficiently.	Remain still and do not talk during a measurement. If "E2" appears repeatedly, inflate the arm cuff manually until the systolic pressure is 30 to 40 mmHg above your previous readings. Refer to the end of instruction manual 2.
	The systolic pressure is above 210 mmHg and a measurement cannot be taken.	
E3 appears	The arm cuff is inflated exceeding the maximum allowable pressure.	Do not touch the arm cuff and/or bend the air tube while taking a measurement. If inflating the arm cuff manually, refer to the end of instruction manual 2.

E4 appears	Moving or talking during a measurement results in vibrations that disrupt the measurement.	Remain still and do not talk during a measurement.
E5 appears	The pulse rate is not detected correctly.	Apply the arm cuff correctly, then take another measurement. Refer to section 6 of instruction manual 2. Remain still and sit correctly during a measurement.
Er appears	The monitor has malfunctioned.	Press the [START/STOP] button again. If "Er" still appears, contact your OMRON retail outlet or distributor.
E rr appears	The monitor cannot connect to a smart device or transmit data correctly.	Follow the instructions shown in the "OMRON connect" app. If the "Err" still appears after checking the app, contact your OMRON retail outlet or distributor.

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
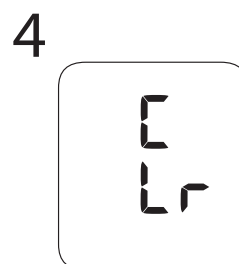
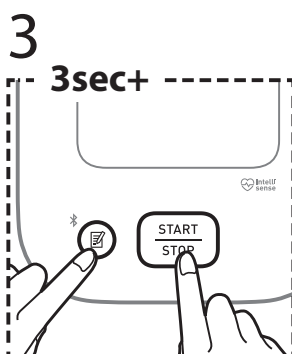
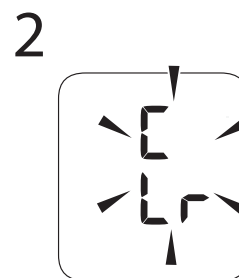
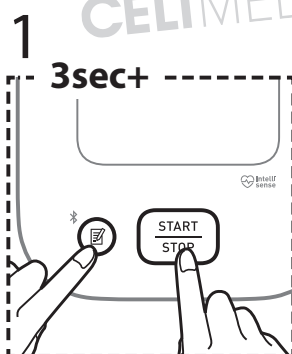
Vraćanje na zadane postavke

CELMED

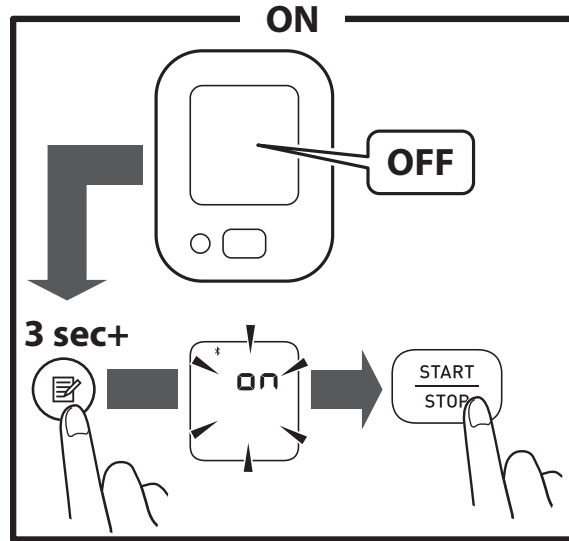Bluetooth is enabled by default. ✖ off appears while Bluetooth is disabled.

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 appears		Apply the arm cuff correctly, then take another measurement. Refer to section 6 of instruction manual 2. Remain still and sit correctly during a measurement. If the irregular heartbeat symbol
 appears	The pulse rate is not detected correctly.	“  ” continues to appear, it is recommended that you consult with your physician.
 does not flash during a measurement		
 flashes	The monitor is waiting for pairing with the smart device.	Refer to section 5 of instruction manual 2 for pairing your monitor with your smart device, or press the [START/STOP] button to cancel pairing and turn your monitor off.
 flashes	- More than 24 readings are not transferred. - Your monitor is not paired with a smart device. - The batteries were replaced.	Pair or transfer your readings to the “OMRON connect” app so you can keep them in memory in the app, and this symbol disappears.
 flashes	Batteries are low.	Replacing all 4 batteries with new ones is recommended. Refer to section 4 of instruction manual 2.

 flashes	Batteries are low.	Replacing all 4 batteries with new ones is recommended. Refer to section 4 of instruction manual 2.
 appears, or the monitor is turned off unexpectedly during a measurement.	Batteries are depleted.	Immediately replace all 4 batteries with new ones. Refer to section 4 of instruction manual 2.
Nothing appears on the display of the monitor.	Battery polarities are not properly aligned.	Check the battery installation for proper placement. Refer to section 4 of instruction manual 2.
Readings appear too high or too low.	Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review sections 2, 6 and 7 of instruction manual 2.	
Any other communication issue occurs.	Follow the instructions shown in the smart device, or visit the “Help” section in the “OMRON connect” app for further help. If the problem still persists, contact your OMRON retail outlet or distributor.	
Any other problem occurs.	Press the [START/STOP] button to turn the monitor off, then press it again to take a measurement. If the problem continues, remove all batteries and wait for 30 seconds. Then re-install the batteries. If the problem still persists, contact your OMRON retail outlet or distributor.	

EN

2 Limited Warranty

Thank you for buying an OMRON product. This product is constructed of high quality materials and great care has been taken in its manufacturing. It is designed to give you a high level of comfort, provided that it is properly operated and maintained as described in the instruction manual.

This product is warranted by OMRON for a period of 5 years after the date of purchase. The proper construction, workmanship and materials of this product is warranted by OMRON. During this period of warranty OMRON will, without charge for labour or parts, repair or replace the defect product or any defective parts.

The warranty does not cover any of the following:

- A. Transport costs and risks of transport.
- B. Costs for repairs and / or defects resulting from repairs done by unauthorised persons.
- C. Periodic check-ups and maintenance.
- D. Failure or wear of optional parts or other attachments other than the main device itself, unless explicitly warranted above.
- E. Costs arising due to non-acceptance of a claim (those will be charged for).
- F. Damages of any kind including personal caused accidentally or from misuse.
- G. Calibration service is not included within the warranty.
- H. Optional parts have a one (1) year warranty from date

of purchase. Optional parts include, but are not limited to the following items: cuff and cuff tube.

Should warranty service be required please apply to the dealer whom the product was purchased from or an authorised OMRON distributor.

For the address refer to the product packaging / literature or to your specialised retailer. If you have difficulties in finding OMRON customer services, visit our website (www.omron-healthcare.com) for contact information.

Repair or replacement under the warranty does not give rise to any extension or renewal of the warranty period.

The warranty will be granted only if the complete product is returned together with the original invoice / cash ticket issued to the consumer by the retailer.

3 Maintenance

3.1 Maintenance

To protect your monitor from damage, follow the directions below: Changes or modifications not approved by the manufacturer will void the user warranty.

⚠ Precaution

DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.

3.2 Storage

- Store your monitor and other components in a clean, safe location.

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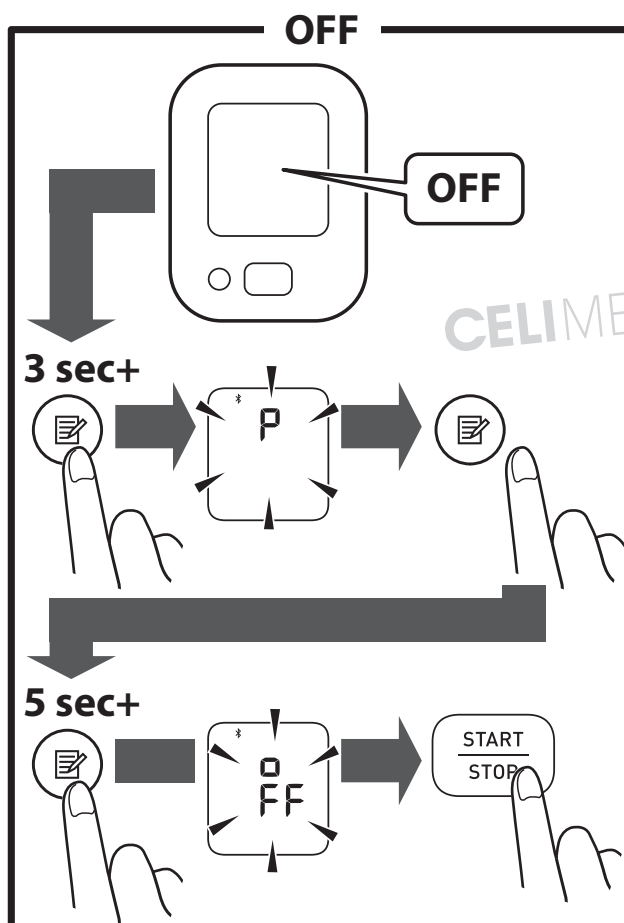
Omogoča shranjevanje do 30 meritev.

Pohranjuje do 30 očitjanja.

Pressing and holding will scroll through the previous reading rapidly.

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- Gently fold the air tube into the arm cuff. Note: Do not bend or crease the air tube excessively.
- Do not store your monitor and other components:
 - If your monitor and other components are wet.
 - In locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.
 - In locations exposed to vibrations or shocks.

3.3 Wiping the Monitor

- Do not use any abrasive or volatile cleaners.
- Use a soft dry cloth or a soft cloth moistened with mild (neutral) detergent to wipe your monitor and arm cuff, and then wipe them with a dry cloth.
- Do not wash or immerse your monitor and arm cuff or other components in water.
- Do not use gasoline, thinners or similar solvents to wipe your monitor and arm cuff or other components.

3.4 Calibration and Service

- The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life.
- It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised OMRON dealer or the OMRON Customer Service at the address given on the packaging or attached literature.

4 Specifications

Product Category	Electronic Sphygmomanometers
Product description	Automatic Upper Arm Blood Pressure Monitor
Model (Code)	M2+ Connect (HEM-7188T1-LE)
Display	LCD digital display
Cuff pressure range	0 to 299 mm Hg
Blood pressure measurement range	SYS: 60 to 260 mm Hg DIA: 40 to 215 mm Hg
Pulse measurement range	40 to 180 beats / min.
Accuracy	Pressure: ± 3 mm Hg Pulse: ± 5 % of display reading
Measurement method	Oscillometric method
Transmission method	Bluetooth® Low Energy
Wireless communication	Frequency range: 2.4 GHz (2400 - 2483.5 MHz) / Modulation: GFSK Effective radiated power: < 20 dBm
Operation mode	Continuous operation
IP classification	Monitor: IP21 Optional AC adapter: IP21 (HHP-CM01 / HHP-AM01) or IP22 (HHP-BFH01)

EN

Rating	DC6 V 4W
Power source	4 "AA" batteries 1.5 V or optional AC adapter (INPUT AC 100–240 V 50–60 Hz 0.12 - 0.065 A)
Battery life	Approx. 900 measurements (Using new alkaline batteries and included arm cuff. Depending on the type of battery and arm cuff.)
Durable period (Service life)	Monitor: 5 years or the time when reaching 30000 times of use. / Cuff: 5 years or the time when reaching 10000 times of use. / Optional AC adapter: 5 years
Operating conditions	+10 to +40 °C / 15 to 90 % RH (non-condensing) / 800 to 1060 hPa
Storage / Transport conditions	-20 to +60 °C / 10 to 90 % RH (non-condensing)
Weight	Monitor: approx. 260 g (not including batteries) Arm cuff for HEM-7188-LE: approx. 170 g Arm cuff for HEM-7188-E: approx. 110 g
Dimensions	Monitor: approx. 98 mm (W) × 79 mm (H) × 138 mm (L) Arm cuff approx. 145 mm × 594 mm (air tube: 750 mm)

Cuff circumference applicable to the monitor	17 to 42 cm (included arm cuff: 22 to 42 cm)
Memory	Stores up to 30 readings
Contents	Monitor, arm cuff (HEM-RML31), 4 "AA" batteries, Instruction Manual 1 and 2
Protection against electric shock	Internally powered ME equipment (when using only batteries) Class II ME equipment (when using optional AC adapter)
Applied part	Type BF (arm cuff)

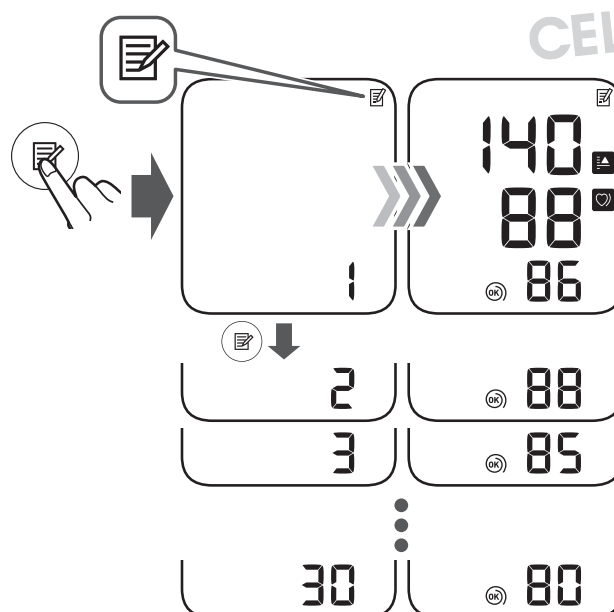
Note

- These specifications are subject to change without notice.
- This monitor is clinically investigated according to the requirements of EN ISO 81060-2:2014 and complies with EN ISO 81060-2:2019+A1:2020+A2:2024 (excluding pregnant and pre-eclampsia patients). In the clinical validation study, K5 was used on 85 subjects for determination of diastolic blood pressure.
- IP classification is degrees of protection provided by enclosures in accordance with IEC 60529. This monitor and optional AC adapter are protected against solid foreign objects of 12.5mm diameter and greater such as a finger. The monitor and optional AC adapter HHP-CM01 and HHP-AM01 are protected against

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*** The high blood pressure definition is based on the 2023 ESH Guideline.**

**** An irregular heartbeat rhythm is defined as a rhythm that is 25 % less or 25 % more than the average rhythm detected while your monitor is measuring blood pressure.**

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vertically falling water drops which may cause issues during a normal operation. The optional AC adapter HHP-BFH01 is protected against oblique falling water drops which may cause issues during a normal operation.

- Operation mode classification complies with IEC 60601-1.
- This monitor communicates with a smart device using Bluetooth Low Energy. Pairing requires user interaction and the transmitted data is encrypted.

About a wireless communication interference

The Bluetooth option in the product is used to connect to dedicated apps on mobile devices to synchronize date/time data from mobile device to the product, and to synchronize measurement data from the product to mobile device. Further handling of the data on the mobile device is up to the user's discretion. This product operates in an unlicensed ISM band at 2.4 GHz where any third party can intercept the radio waves, willfully or accidentally, for any unknown purpose. In the event this product is used near other wireless devices such as microwave and wireless LAN, which operate on the same frequency band as this product, there is a possibility that interference may occur. If interference occurs, stop the operation of the other devices or relocate this product away from other wireless devices before attempting to use it.

5 Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.

6 Important Information regarding Electromagnetic Compatibility (EMC)

This device conforms to the EN 60601-1-2:2015+A1:2021 Electromagnetic Compatibility (EMC) standard. Further documentation in accordance with this EMC standard is available at <https://healthcare.omron.com/electro-magnetic-compatibility>. Refer to the EMC information for this device on the website.

7 Guidance and Manufacturer's Declaration

- Hereby, OMRON HEALTHCARE Co., Ltd., declares that this device is in compliance with Directive 2014/53/EU.
- The full text of the EU declaration of conformity is available at the following internet address: www.omron-healthcare.com
- This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., Japan. The Core component for OMRON blood pressure monitors, which is the Pressure Sensor, is produced in Japan.
- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

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